Nikon

読書用ルーペ/Reading Magnifier

JP

EN

U1-4D

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THANK YOU FOR PURCHASING THE NIKON READING MAGNIFIER.

- Thoroughly read this instruction manual before using the product, and be sure to use the product correctly.
- After reading, retain this instruction manual so that it can be easily referenced.
- Specifications and design are subject to change without notice.
- No reproduction in any form of this manual, in whole or in part (except for brief quotation in critical articles or reviews), may be made without written authorization from NIKON VISION CO., LTD.

SAFETY PRECAUTIONS

STRICTLY OBSERVE THE FOLLOWING FOR SAFETY

■ The following marks are used to indicate the degrees of injury or damage that could result from misuse of this product.

 WARNING	This indication alerts you to the fact that any improper use ignoring the contents described herein can result in potential death or serious injury.
 CAUTIONS	This indication alerts you to the fact that any improper use ignoring the contents described herein can result in potential injury or may cause only a material loss.


WARNING

- Never look at the sun directly through this product.

CAUTIONS

- Do not leave this product in direct sunlight.
- Do not look at things through this product while walking.
- Do not leave the package for this product (or packing material, accessories, etc.) within small children's reach.

Overview

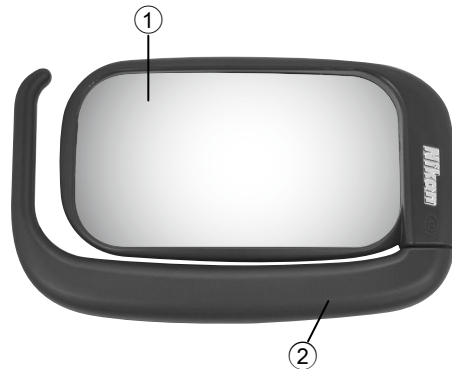U1-4D

- ① Lens
- ② Rotating handle

Reading Magnifier U1 series features

- The lens is made of a highly transparent acrylic (PMMA), and features a hard coating on its surface to prevent scratching.
- There is no difference between the front and rear of the lens. (The lens used is an equiconvex aspherical lens.) The Reading Magnifier can be held in your left or right hand.

(Holding with your left hand)

(Holding with your right hand)


- Our “rotating handle’s” universal design makes it easy to grip while preventing slipping, and enables you to use it as a stand to support your hand, reducing the burden on your arm.

Usage

- Remove the protective film before using this product for the first time.
- To use the Reading Magnifier, rotate the closed rotating handle (Fig. 1). You can rotate the rotating handle 360° and use it at your preferred position. The rotating handle also clicks at 45° increments (Fig. 2).

(Fig. 1)

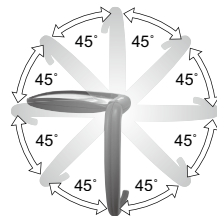(Fig. 2)

*The correct position for the lens depends on your visual acuity.

*If you use glasses or contact lenses, use the Reading Magnifier with those corrections to your visual acuity.

- The rotating handle can also be used as a stand to support your hand (Fig. 3).

(Fig. 3)

About cleaning

- Do not wash this product with cold or hot water.
- Gently wipe off dirt from this product (except for the lens) by using a soft, clean cloth. Do not use organic solvents such as benzene, thinner, or any cleaning solution that includes organic solvents, etc.
- Remove dust from the faces of the lens with a soft, non-oily brush.
- Remove fingerprints, etc., from the faces of the lens by gently wiping them with cleaning paper or a soft cleaning cloth that has been dampened with a small amount of anhydrous alcohol (ethanol) or commercially available lens cleaner.

Specifications

	4D (Square)
Effective diameter of lens (mm/in)	100 × 54/3.9 × 2.1
Refractive power (diopters)	4
Reference magnification*	1.5x
Lens material (surface process)	Acrylic (PMMA) lens (with hard coating)
Lens form	Equiconvex aspherical lens
Size (mm/in) (LxWxD)	83 x 142 (up to 242 when the handle is opened) x 18
Weight (g/oz)	103

* Reference magnification is when an object is clearly visible at approx. 250 mm/9.8 in.